


Innovation for all

AGUETTANT® SELF INJECTOR PEN

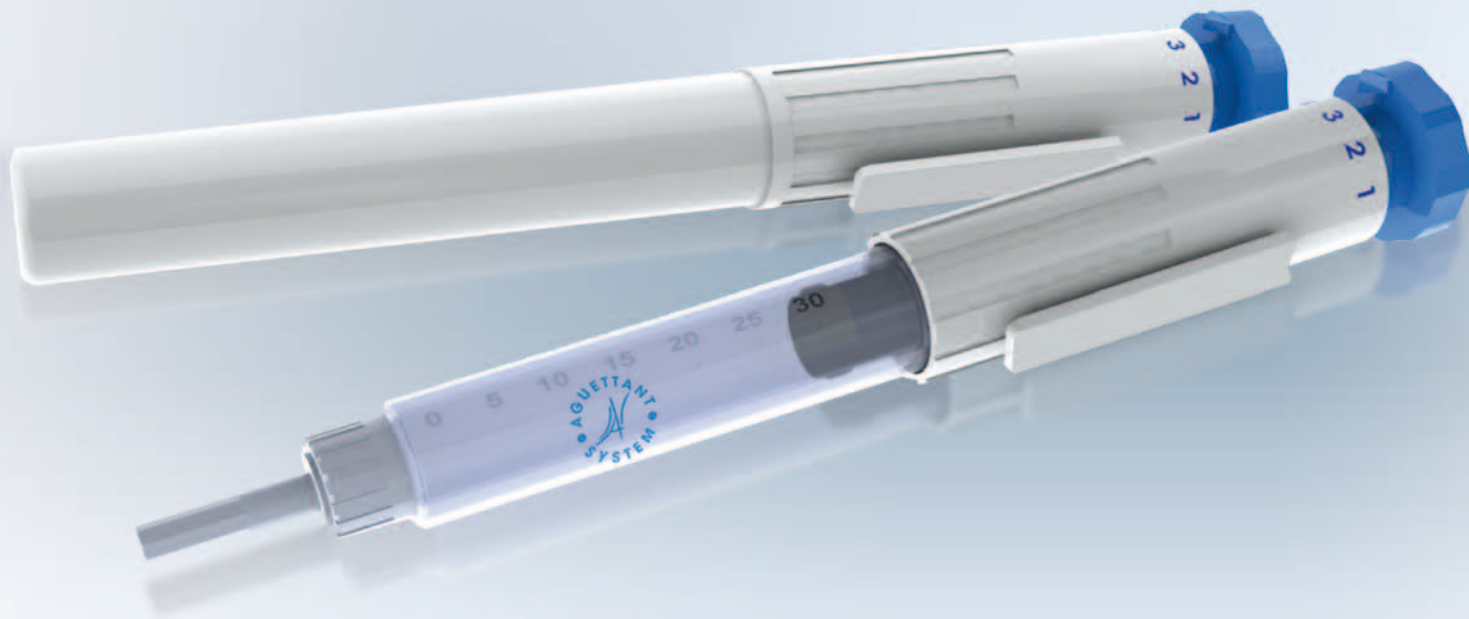

| A UNIQUE READY TO USE AND MULTIDOSE SELF-INJECTOR PEN |





AGUETTANT
working with you everyday

AGUETTANT® SELF INJECTOR PEN

Ready to use multidose Self-Injector Pen: A safe solution for subcutaneous self-injection

The Aguettant self-injector pen allows the patient to easily select the required dose and to safely self inject. Moreover, this pen is ready to use and can provide up to 30 injections. Another great feature is the memorization of the last injected dose. The pen is disposable once the cartridge is empty.

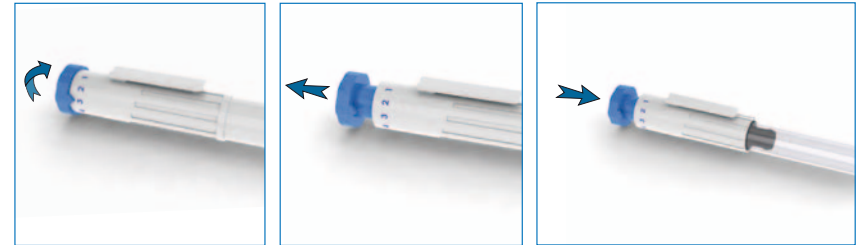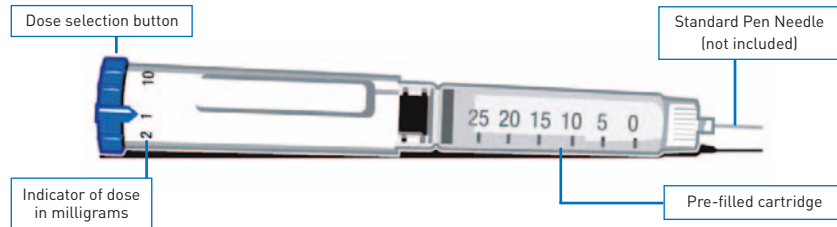

Select the dose

Press down the dose selection button
Turn clockwise to select the required dose

Pull to arm

Push to inject

Technical Specifications:

- **Volume capacity per dose: from 0.1 ml to 1 ml**
Development of the pen possible to reduce to 0.05 ml
- **Dose accuracy: -0% to +5% for 0,1ml**
- **Number of increments possible: from 1 to 10**
Development of the pen possible to increase up to 16
- **Cartridge contents: 3 ml and compatible with 1,8ml cartridge**

A patented concept:
US 5807346
EPO 773806

Benefits for the patients:

- **Safe:** Protective overall body allows to carry the pen without risks of injury. For improved monitoring, the volume of drug left is always visible through the translucent cartridge and displayed on the plunger rod. The selection button memorizes the last dose injected
- **Efficient and Reliable:** Dose delivery is highly precise to 5µl
- **Robust:** The pen cannot be disassembled by the patient
- **Ergonomic :** No assembly is required. The dosage is easy to select and the handling of the pen is comfortable
- **Multi Use Feature:** From 30 to 60 injections per pen, depending on the minimum dosage design
- **Compatible:** To be used with standard pen needles
- **Commercially proven:** Marketed in Europe for more than 15 years



Solutions for preserving life

www.aguettant.com

Laboratoire **AGUETTANT**
Headquarters & Lyon manufacturing plant
1 Rue Alexander Fleming - 69007 Lyon
France