
Easyflex⁺

YZG0119EU
YZG0219EU
YZG0419EU
YZG0719EU

REF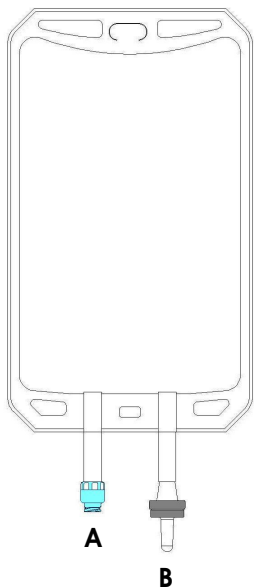

Ne pas utiliser d'aiguille !
Do not use a needle !
Keine Nadel gebrauchen !
Geen naalden gebruiken !

FR

POCHE VIDE
SANS PVC

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EN

PVC FREE
EMPTY BAG

Polyolefin empty bag with needleless access port and self-sealing infusion site.

DESCRIPTION

The Easyflex⁺ empty bag has an infusion site with a self-sealing membrane protected by a break-off tip and a secure needleless additive port with a self-resealing valve. Connection onto the valve is done with a male Luer-lock (coming from a Luer syringe or infuser). Latex free - DEHP free.

INDICATIONS AND PERFORMANCE

Empty bag for the preparation of a solution for infusion. The Easyflex⁺ empty bag allows secure reconstitution with needleless access. Sealed and integrated empty bag with self-resealing valve (no leaks). The duration of use of the device depends on the solution contained in the bag. Refer to the SmPC of drugs.

INSTRUCTIONS FOR USE

- Check the expiry date on the label.
- If the overwrap or bag are damaged: do not use.
- Only remove the bag from the overwrap when ready to use.
- Peel off the overwrap.
- Put the bag on a clean surface.
- Connect the male Luer-lock from a syringe to the needleless access port of the bag (A) (compliant with ISO standard 594-2) by pressing and turning the syringe clockwise.
- Transfer the solution to the bag.
- Disconnect the syringe by turning counterclockwise.
- Release the trocar access part (B) of the infusion site by breaking the break-off tip with your thumb and index finger.
- Introduce the bag (compliant with standard ISO 8536-4) into the infusion site up to the guard with a linear movement (no rotation).
- After use, discard the device according to the current protocol in the facility.

WARNINGS AND PRECAUTIONS

- This device is a sterile product, for single use: do not resterilize.
- Before adding any drugs, check the compatibility with polyolefins. Refer to the SmPC of drugs.
- The bags are sterilized by beta radiation and may be slightly discoloured; this is not a visible sign of damage.

CONTRAINDICATIONS

No contraindications for use of the device.

ADVERSE REACTIONS

No known adverse reactions related to the use of the device.

STORAGE CONDITIONS

- Keep away from light, humidity and freezing at a temperature below 25°C.

REUSE

Risk related to reuse: septic risk.

MATERIOVIGILANCE AND TRACEABILITY

This product is falls within the scope of materiovigilance. Any incident or adverse reaction related or possibly related to the use of the product must be reported to the corresponding materiovigilance location for the facility in which the product is used.

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